

THE GOAL

Make The Moses™ appliance last. Keep it clean and hygienic.

OVERVIEW

The Moses™ is a two piece open-anterior device. The upper element is a .3mm polypropylene-ethylene copolymer vacuum-formed splint covering all maxillary teeth and trimmed to avoid impinging on the gingiva. The lower element is made of methyl-methylacrylate dental acrylic resin.

Warpage of acrylic resin occurs from allowing the appliance to dry out after it is processed. Completed acrylic resin prostheses of any kind should be stored or immersed in a container of clean water to prevent them from drying out, and hence the appliance can be dimensionally more stable.

To clean the appliance for not more than ½ minute with a brush before and after use, such as a tooth brush just for this purpose, is a good idea to keep it clean and hygienic. The concern of scratches on the acrylic surface caused by brushing should be minor as it may take years for small scratches to appear.

"Kleenite" from Regent Labs is the recommended denture cleaner to use with The Moses™ appliance. It is not readily sold in all areas of the US, but available online. Other denture cleaners such as Polident and Efferdent are individually wrapped tablets that are easy to travel with.

To store the Moses™ appliance in water when not in use and regularly clean it slightly with a brush is all we need to keep the appliance in a clean, hygienic and good working condition. All Moses™ appliances are strongly built with premium acrylic resin and do not break from bruxism. Under normal use the appliance can last for a long time.

Careful not to drop your appliance, It can break. And keep away from children and animals, dogs love to chew oral appliances.

DAILY

- Rinse in cold or warm water
- Clean with a toothbrush, briefly, for about 30 seconds
- Do not use toothpaste as some brands can scratch and damage the surface.
- Completely immerse in a container of clean water.

TWICE A WEEK

- Soak in a partial denture cleaner, per instructions on box
- We recommend "Kleenite" from Regent Labs
- Sold online and at most drug stores.

